ONE TOUCH® UltraSoft

Lancets/Kim lấy máu

AW 07294701A

OneTouch® UltraSoft Lancets are for use with all OneTouch® Penlet® and OneTouch® UltraSoft brand automatic blood samplers from LifeScan, and most other blood sampling devices.

Intended use


The OneTouch® UltraSoft lancets are sterile, single use devices that have been designed for single patient use in a general environment. Their intended use is for performing skin punctures on adults, children and neonates for the express purpose of obtaining blood samples for a variety of reasons e.g. diabetic blood glucose monitoring.

However, frequency of use and intended patient population is dependent on the treatment regime.


The OneTouch® UltraSoft lancet is compatible with the single patient use OneTouch® UltraSoft and OneTouch® lancing devices.

Instructions for Use


The end user loads the lancet into the lancing device.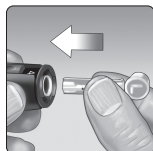


Twist off the moulded cap to expose the sterile needle immediately prior to use. Save the cap for later use.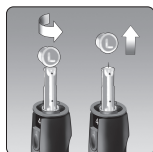

After preparing the test site, the subject will follow the lancing device instructions for use to perform a lancing event on the appropriate finger.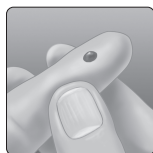

The used lancet should be capped with the moulded cap and disposed of in accordance with local requirements

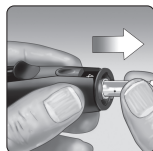Disposing of the used lancet

It is important to discard the used lancet carefully after each use to avoid unintended lancet stick injuries.

Contraindications

The OneTouch® UltraSoft lancets are not indicated to perform Alternate Site Testing (AST) and should not be used to test on the forearm or palm.

Warnings and precautions

- For single use only.
- For your safety, never share the lancet.
- Use the convenient snap-on cap for safe disposal.

- **Do not** use if protective disk has been damaged or removed.
- **Do not** purchase if seal at either end is broken or missing.

Packaging size

Box of 25 lancets.

Box of 100 lancets.

Storage

Store between 5°C–30°C.

Shelf life

5 years from the manufacturing date.

OUR COMMITMENT TO YOU:

Our goal is to provide you with quality healthcare products and dedicated customer service. If you are not really satisfied with this product or if you have questions about the use of any LifeScan product, contact OneTouch® Customer Service at:

In Vietnam:

Freephone: 1800 599 915.

Working Hours: 8:30-17:30.

Description of symbols

For a complete description of all symbols used, please refer to the User's Guide that came with your system.

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© 2018–2022 LifeScan IP Holdings, LLC
Asahi Polyslider Company, Limited
LifeScan Europe GmbH